

[CLICK TO LEARN MORE](#)

The benchmark in nasal pillows masks

Engineered for performance.

AIRFIT™ P10



Defining excellence for over a decade

For more than a decade, providers have relied on AirFit™ P10 to help patients start therapy and stay on it. Today, strong compliance outcomes and HME validation continue to reinforce its position as the benchmark in nasal pillows masks. From its secure dual-wall seal to its ultra-quiet performance and simple design, AirFit™ P10 continues to deliver the consistent experience providers have trusted for years.¹

#1

choice of HMEs across comfort, seal and durability.¹

90%

patient compliance in just 39 days.²

1.9 L/min

lower leak rates than the average for pillows masks.³

Simple by design


With just three parts, AirFit™ P10 is simple to fit, put together, clean and maintain.

- ① Headgear ② Cushion ③ Frame and short tube



The same proven performance, Designed for her

AirFit™ P10 for Her offers the same performance in a design tailored for women, with a smaller size range and headgear styled for a more personalized fit.

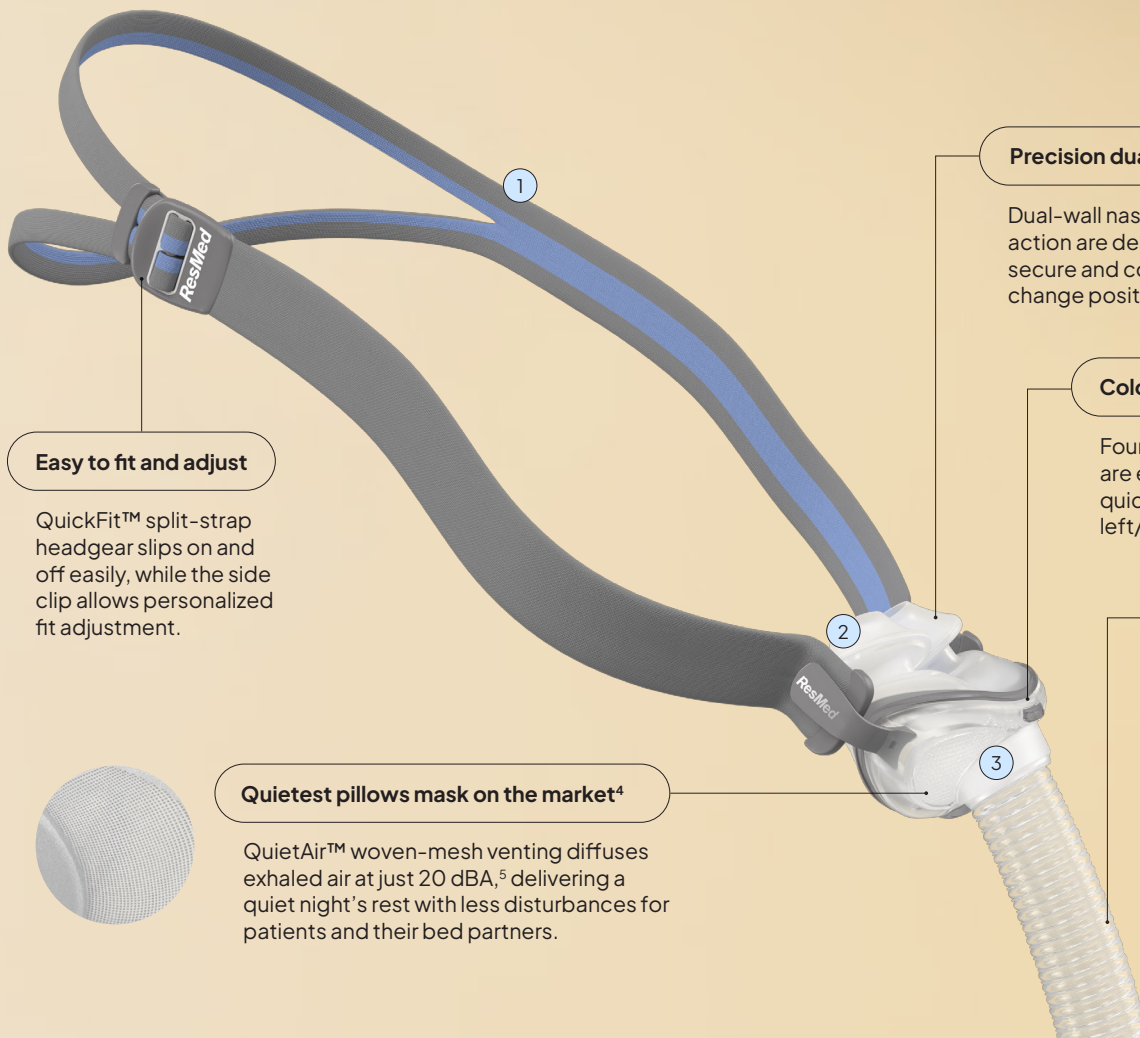

Easy to fit and adjust

QuickFit™ split-strap headgear slips on and off easily, while the side clip allows personalized fit adjustment.

Precision dual-wall seal

Dual-wall nasal pillows with trampoline action are designed to maintain a secure and comfortable seal as patients change position during sleep.

Color-coded nasal pillows


Four pillow sizes (XS, S, M, L) are each color-coded for quick size identification and left/right orientation.

Lightweight flexi-tube

Designed to provide stability while allowing patients to move naturally during sleep.

Quietest pillows mask on the market⁴

QuietAir™ woven-mesh venting diffuses exhaled air at just 20 dBA,⁵ delivering a quiet night's rest with less disturbances for patients and their bed partners.

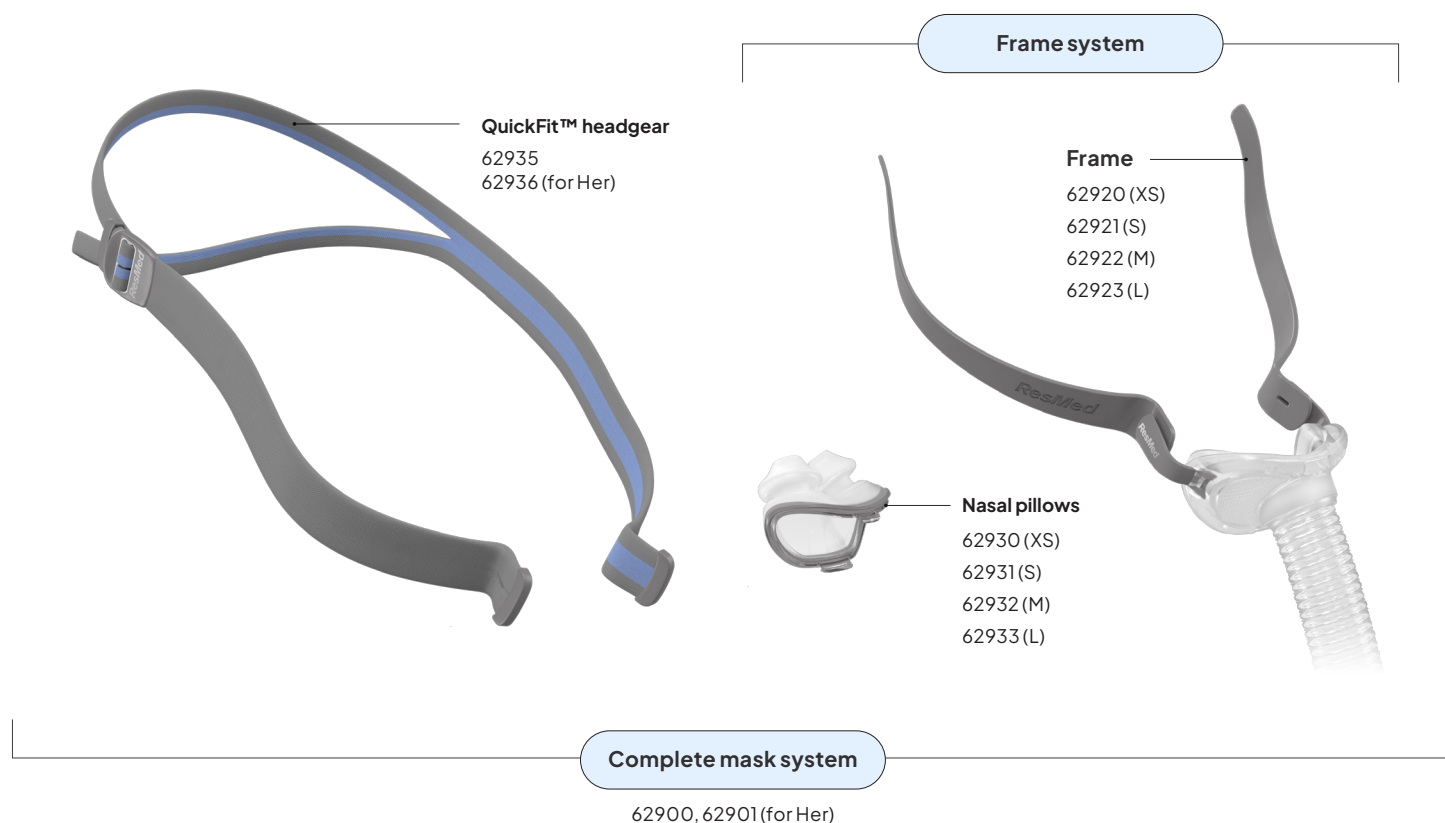

Product codes

Complete mask system

(includes mask frame, three pillow sizes* and headgear)

Product name	Product code
AirFit P10 mask system	62900
AirFit P10 for Her mask system	62910

*AirFit P10 mask system includes S, M and L pillows. AirFit P10 for Her mask system includes XS, S and M pillows.

Pillows

Product name	Product code
AirFit P10 pillows - XS	62930
AirFit P10 pillows - S	62931
AirFit P10 pillows - M	62932
AirFit P10 pillows - L	62933

Pillow sizes: Extra small (XS), Small (S), Medium (M), Large (L)

Frame system

(includes mask frame and cushion)

Product name	Product code
AirFit P10 frame system - XS	62920
AirFit P10 frame system - S	62921
AirFit P10 frame system - M	62922
AirFit P10 frame system - L	62923

Headgear

Product name	Product code
AirFit P10	62935
AirFit P10 for Her	62936

Fitting guide



While holding the pillows away from your nose, make sure the pillow size and left (L) and right (R) indicators are facing towards you.



Guide the nasal pillows into your nostrils. Pull the QuickFit™ headgear over the head by holding the bottom band and stretching it around the back of the head. The top band should sit comfortably on the top of your head.



To achieve a comfortable fit, spread the split bands on the QuickFit™ headgear apart to loosen the mask or draw them closer together to tighten the mask.



Briefly pull the mask away from your face, then place it back. Ensure the nasal pillows fit securely in your nostrils and that both pillows are positioned upright.



To adjust the QuickFit™ headgear, pull the band through the single side-clip until you achieve a comfortable seal. Do not overtighten and readjust as needed over time.



Connect the mask tube to the device — your mask is now ready to use.



For longer hair, we recommend feeding it through the back of the headgear. If tied in a ponytail, it should sit high enough to fit comfortably through the split band on the headgear.

Refer to the user guide for further instructions and complete labeling information including indications, contraindications, warnings and precautions if applicable.

¹ Based on an online, double-blinded survey of 92 U.S.-based HME providers conducted by Peel Research Partners, Dec 12–22, 2025.

² Data are derived from an analysis of new PAP therapy patients in the United States using AirView™ and myAir™ platforms. Patients were included if myAir™ setup occurred from 7 days before through 14 days after the AirView™ setup date (i.e., –7 to +14 days) and valid mask data were available. Compliance (90-day) was assessed for setups between January 2024 and December 2024. Compliance was defined as PAP use for ≥4 hours per night on ≥70% of nights during any consecutive 30-day period within the first 90 days of therapy, with the AirView™ setup date counted as day 1. Days to compliance represents the number of days from AirView™ setup until the first date the patient achieved compliance. Analysis included the top 45 mask models, representing 99.8% of mask selections among new users in myAir™. Days to compliance (90th percentile) is the number of days from AirView™ setup by which 90% of patients achieved compliance. Overall average = 41.7 days mean across the 45 included mask models; n = 557,021. AirFit P10 = 39 days, n = 65,796.

³ Data are derived from an analysis of U.S. PAP therapy patients using AirView™ and myAir platforms. Patients were included based on registration within ±14 days of AirView setup and valid mask data. Compliance was assessed for 2024 setups (n = 557,021; AirFit™ P10 n = 65,796), and 1-year adherence for 2023 setups (n = 389,270; AirFit™ P10 n = 50,156). Compliance was defined as PAP use for ≥4 hours per night on ≥70% of nights during any consecutive 30-day period within the first 90 days of therapy. AirFit™ P10 demonstrated lower leak rates than the average for pillows masks (median 1.9 vs. 2.2 L/min; 95th percentile 12.0 vs. 12.7 L/min). No statistical significance testing was performed.

⁴ Leading nasal pillows masks selected based on Resmed analysis of U.S. market data as of June 2026. Masks reviewed: F&P Nova Micro, F&P Solo Pillows, Philips DreamWear Pillows, React Health Rio II, and AirFit P10. Comparison based on manufacturer-published user-guide sound power declarations and Resmed objective bench testing comparing AirFit P10 and F&P Nova Micro Pillows under flow-normalised conditions (approx. 33 L/min).

⁵ Resmed internal verification testing conducted June 2026.