



NASAL PILLOWS MASK

**Designed for Confidence.
Built for Comfort.**

NEW

Gaining patient confidence is key to successful CPAP therapy and patient confidence begins with a mask that fits and is comfortable.



Presented With Compliments From
www.DirectHomeMedical.com

Fisher & Paykel
HEALTHCARE

F&P Brevida™ Key features and benefits

Adjustable Headgear

Adjustable headgear which is designed to stay in place during sleep.

AirPillow™ Seal

The innovative AirPillow™ inflates to form a gentle and effective seal.




VisiBlue™

Blue highlights incorporated into the mask key components assist patients with daily mask use.

Washable Diffuser

The diffuser, which reduces noise and air draft, is washable and durable.

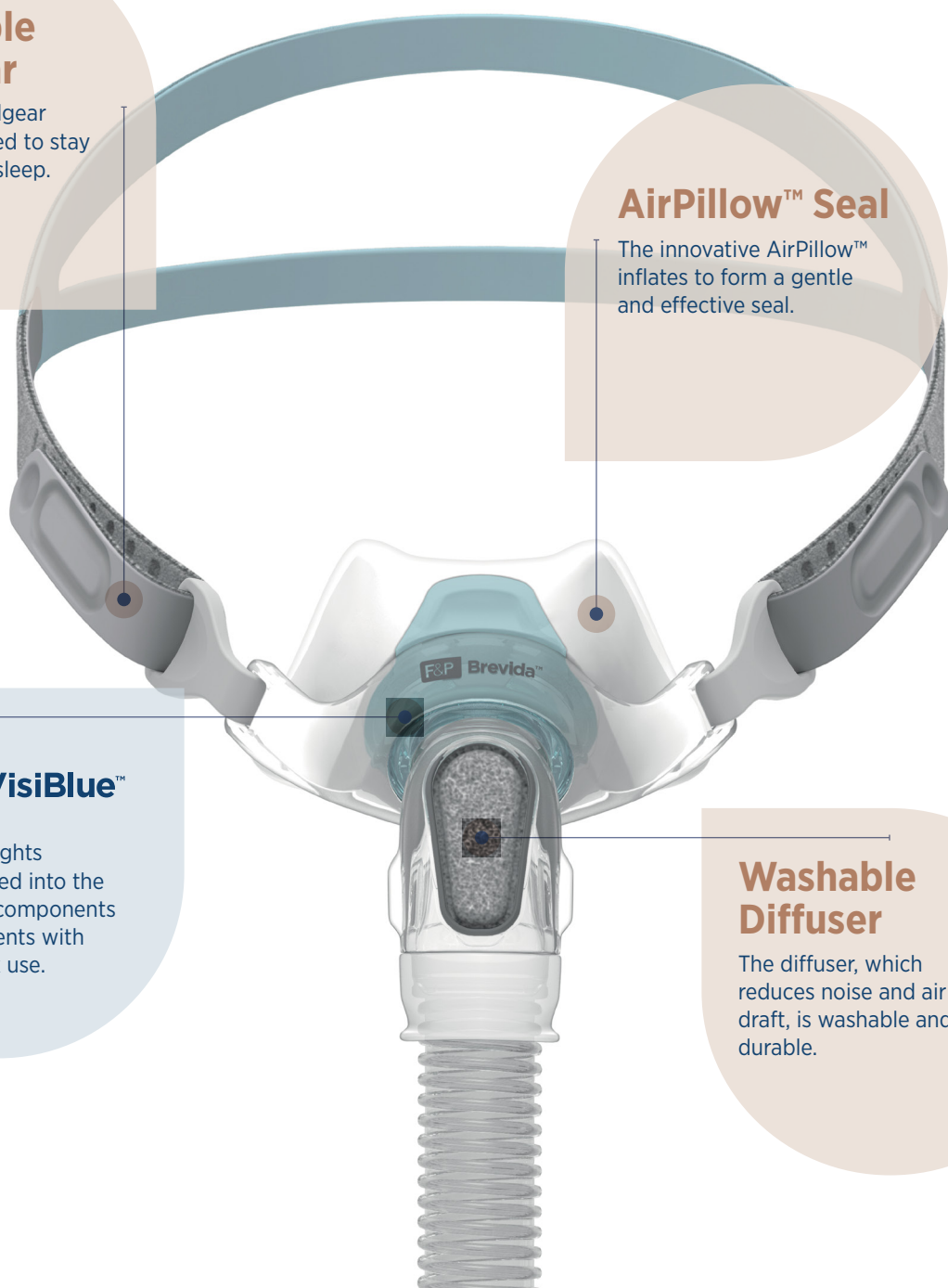

The innovative F&P Brevida™ AirPillow™ Seal

Two seal sizes

Extra Small-Small and Medium-Large.

Extra-thin, soft seal


Sealing region made of extra-thin (0.25 mm) soft silicone to provide a gentle seal.

Versatile pillows

Versatile pillow geometry to fit a wide range of nostril sizes.

Stabilizing side supports

Side supports designed to reduce the likelihood of the seal dislodging during sleep.

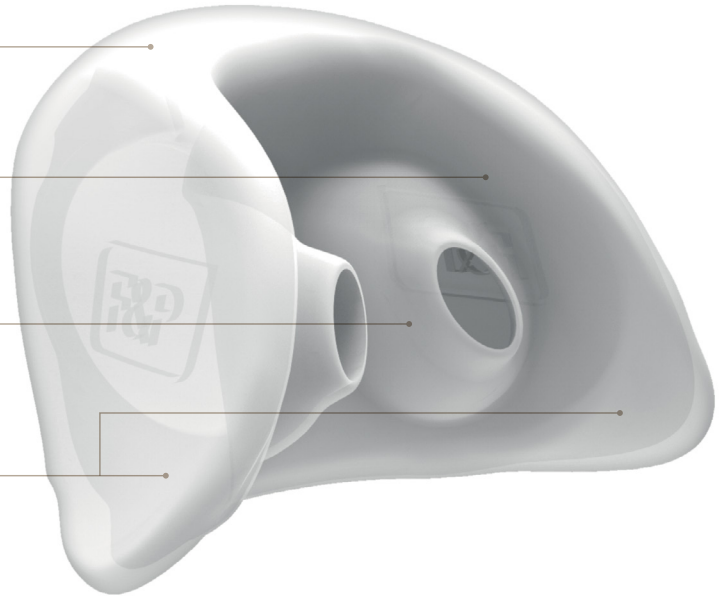


A gentle, soft seal

The AirPillow™ seal inflates to form a gentle ‘pillow’ of air and seals in and around the nose.


F&P AirPillow™ Nasal Pillows Seal

Seals in and around
the nose

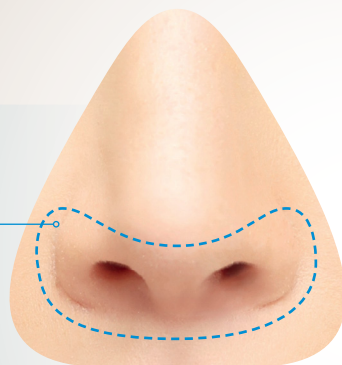

Traditional Nasal Pillows Seal

Seals directly in the
nostrils

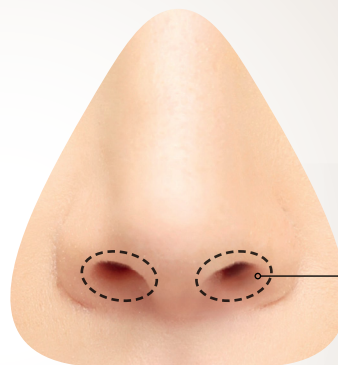

Fitting your F&P Brevida™



Step 1.
Hold the silicone seal away from your nose and use your fingers to carefully spread open the soft side supports of the silicone seal and then guide the two openings into your nostrils.



Step 2.
Hold the mask frame in position with one hand and pull the headgear over your head with the other hand, positioning the straps above the ears as per the fitting image above.



Step 3.
Gently tighten or loosen the headgear straps on the sides of the headgear until you achieve a comfortable seal.

REORDER CODES

MASKS

BREISA	Brevida Extra Small-Small (one XS-S seal)
BREIMA	Brevida Medium-Large (one M-L seal)
BREISMA	Brevida XS-S and M-L (two seals)

SPARE PARTS

400BRE113	Brevida Seal Extra Small-Small
400BRE114	Brevida Seal Medium-Large
400BRE120	Brevida Headgear
400BRE151	Brevida Headgear Clips
400BRE161	Brevida Diffuser
400BRE171	Brevida Elbow
400FPH241	F&P Swivel 2
400BRE131	Brevida Mask No Headgear XS-S
400BRE132	Brevida Mask No Headgear M-L