

QUICK START GUIDE

FreeStyle® Comfort® Backpack

1 FreeStyle Comfort and backpack.



2 Insert FreeStyle Comfort into backpack.



3 Loosen and completely extend straps. Put on backpack and adjust straps.



Note: For optimal comfort, adjust straps so the FreeStyle Comfort is sitting in the middle of your back.

4 Avoid directly removing the backpack. Loosen straps before removing.



→ 🏠 + DirectHome
MEDICAL

[CLICK TO LEARN MORE](#)