

# Fitting the Eson 2



Hold open the bottom straps of the Headgear with the Mask Frame facing downwards.



Guide the headgear over your head so that the silicone seal is resting on your forehead.



Place your hands on the bottom straps on either side of the mask frame and pull out and downwards. Ensure the silicone seal rests over your nose and the blue top strap is sitting on your forehead.



Gently tighten the headgear straps of the mask headgear starting with the blue forehead straps followed by the bottom straps.

## REORDER CODES

### MASKS

ESN2SA	ESON2 Small
ESN2MA	ESON2 Medium
ESN2LA	ESON2 Large

### SPARE PARTS

400ESN214	ESON2 Seal Small
400ESN215	ESON2 Seal Medium
400ESN216	ESON2 Seal Large

400ESN221	ESON2 Headgear Small
400ESN222	ESON2 Headgear Medium-Large
400ESN231	ESON2 Mask No Headgear Small
400ESN232	ESON2 Mask No Headgear Medium
400ESN233	ESON2 Mask No Headgear Large
400FPH241	F&P Swivel 2
400ESN251	ESON2 Clips
400ESN261	ESON2 Diffuser
400ESN271	ESON2 Elbow



**CLICK TO LEARN MORE**