

Disinfection Guide for Professional Use Only

GoLife Nasal Pillows Mask

⚠ Caution

- These cleaning and disinfection instructions are for **Clinical Use** only.
- The following processes are validated for the GoLife nasal pillows mask when used in accordance with the instructions provided by the manufacturer of the cleaning and disinfection products and/or processes. Any deviation from said instructions or the cleaning agents listed below may impact the performance or durability of the product. Review those instructions for additional warnings and cautions.

⚠ Warning

- Inspect all parts for damage or wear; replace any parts that have visibly deteriorated (cracking, crazing, tears, etc).
- The headgear and/or headgear frame cannot be processed using the cleaning and disinfection methods listed. The headgear/headgear frame must be replaced.

📌 Note

- After disinfection, discoloration of the cushion and a slight odor is normal.

Cleaning

- Remove the elbow with integrated tubing from the headgear frame as instructed below.
- Clean the mask using a soft bristle brush to adequately remove adhering substances from each component while soaking in a commercially available, anionic detergent (Example: MediZime LF). Extra attention should be given to the crevices and cavities during cleaning.

📌 **Note:** If compatible with the manufacturer's instructions, the detergent may be applied within the washer/disinfector cycle.

- Rinse the mask with water and air dry out of direct sunlight. If cleaning manually, use a minimum of 5 liters per mask.


Removable Elbow with Integrated Tube

The exhalation elbow with integrated tubing can be removed for cleaning.


1. Remove the cushion (Figure 1).
2. Hold the headgear frame with the elbow between your first and second fingers. With your thumb, push the back of the elbow through the frame (Figure 2).

⚠ **Caution:** Do not pull on the 15mm tubing or damage may occur.


3. Clean per instructions.
4. Reattach the clean elbow with integrated tubing to the frame (Figure 3).
5. Attach the pillows cushion.

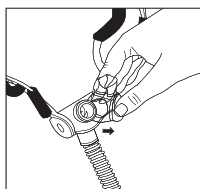

📌 Figure 1

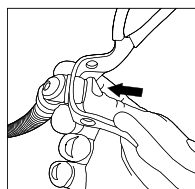

📌 Figure 2

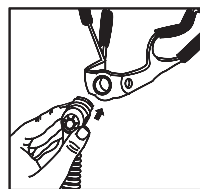

📌 Figure 3



Presented With Compliments From
www.DirectHomeMedical.com

High-Level Disinfection

The following high-level disinfection processes are intended for multi-patient use in a clinical or hospital/institution environment only. For home cleaning or single patient use, refer to the **Instructions for Use** included with the product.

Thermal Disinfection

Temperature	Time	Maximum Cycles
70° C	100 minutes	30

- Rinse thoroughly and air dry out of direct sunlight. Make sure the mask is dry before use.
- Inspect all parts for damage or wear; replace any parts that have visibly deteriorated (cracking, crazing, tears, etc).

Contact Philips Respironics

Visit us on-line at www.philips.com/respironics to download copies of the Disinfection Guide or contact Philips Respironics Customer Service. Respironics: 1-800-345-6443 (USA or Canada only) or 724-387-4000

GoLife is a trademark of Philips Respironics Inc. MEDIZIME LF is a trademark of Medical Chemical Corporation.

Respironics Inc.
1001 Murry Ridge Lane
Murrysville, PA 15668 USA

