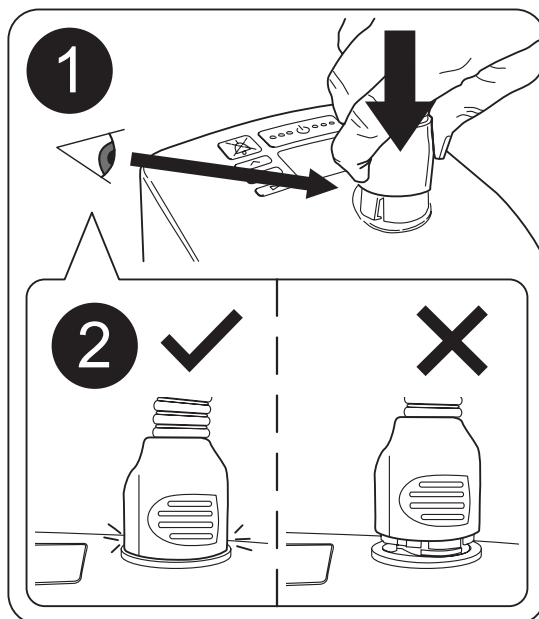
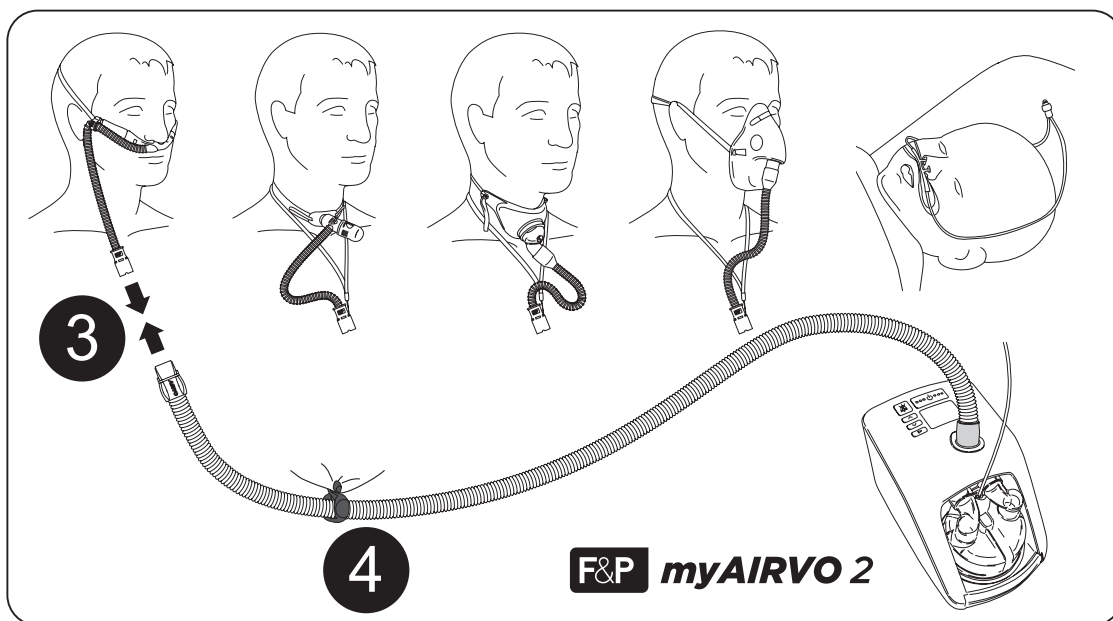
[CLICK TO LEARN MORE](#)

CE 0123 Rx only

F&P, AIRVO and myAIRVO are trademarks of Fisher & Paykel Healthcare Ltd.
For patent information, see www.fphcare.com/ip

Fisher & Paykel
HEALTHCARE

Manufacturer ■ Fisher & Paykel Healthcare Ltd, 15 Maurice Paykel Place, East Tamaki, Auckland 2013, PO Box 14 348, Panmure, Auckland 1741, New Zealand Tel: +64 9 574 0100 Fax: +64 9 574 0158 Email: info@fphcare.co.nz Web: www.fphcare.com

Australia (Sponsor) Fisher & Paykel Healthcare Pty Ltd, 19-31 King Street, Nunawading, Melbourne, Victoria 3131. Tel: +61 3 9871 4900 Fax: +61 3 9871 4998

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USA/Canada Tel: 1800 446 3908 or +1 949 453 4000 Fax: +1 949 453 4001

myAIRVO™ Heated breathing tube 900PT560/900PT560E

(en)

Indications for Use

The 900PT560 and 900PT560E Heated breathing tube is for delivery of humidified respiratory gases. For use with myAIRVO™ and myAIRVO™ 2 Series humidifiers in homes and long-term care facilities.

For use at flows from 2 to 60 L/min depending on the patient interface.

Compatible patient interfaces: nasal cannula, tracheostomy direct connection, mask adapter.

System Specifications

Circuit Length: 1.8 m (6 ft) + interface

Setup

- myAIRVO/myAIRVO 2 humidifier (“myAIRVO”) on a flat surface.
- OPT316/OPT318/OPT942/OPT942E/OPT944/OPT944E/OPT946/OPT946E/OPT970/OPT970E/OPT980/OPT980E patient interface.
- Cautions: Use only USP sterile/distilled water for inhalation, or equivalent.
- This product is not intended for use with any medication.

Warning: Use of a non-approved accessory could impair performance or compromise safety.

Home Use

- This product is intended to be used for a maximum of 60 days for home use, provided that the myAIRVO cleaning and maintenance instructions are followed.
- Do not soak, wash or sterilize.

Step 1

Connect breathing tube to myAIRVO.

Step 2

Make sure blue sliding collar is fully clicked down into place.

Step 3

Connect breathing tube to patient interface once the system has warmed up. Position breathing tube below patient interface so that condensate flows away from the patient.

Step 4

Connect breathing tube clip to patient clothing or bedding. Note: connecting to patient clothing may not be suitable for all patient groups.

WARNINGS

- For single patient use only. Reuse may result in transmission of infectious substances. Attempting to reprocess will result in degradation of materials and render the product defective.
- Using the breathing tube for longer than the specified time can result in serious injury including infection.
- Never operate the unit if the breathing tube has been damaged with holes, tears or kinks.
- Do not block the flow of air through the unit and breathing tube.
- California residents please be advised of the following, pursuant to Proposition 65: This product contains chemicals known to the State of California to cause cancer, birth defects and other reproductive harm. For more information, please visit: www.fphcare.com/prop65

To avoid burns

- Do not allow the breathing tube to remain in direct contact with skin for prolonged periods of time.
- Do not use in the presence of a naked flame, to avoid fires.
- Do not add heat to any part of the breathing tube e.g. covering with a blanket.

Cautions

- Avoid contact with chemicals, cleaning agents, or hand sanitizers.

To prevent condensation

- Use in a room warmer than 18 °C (64 °F).
- Remove/minimize the impact of anything that may cool the heated breathing tube, e.g. a fan, air conditioning, open window.

To manage excessive condensation

- Place the myAIRVO below patient head height.
- Drain condensate back into the water chamber.
 - At higher target flow settings it may be necessary to first reduce the target flow setting to 30 L/min, to ensure that the condensate drains safely and effectively.
- Disconnect the patient interface from the heated breathing tube.
- Lift the patient end of the heated breathing tube, allowing the condensate to run into the water chamber.

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