




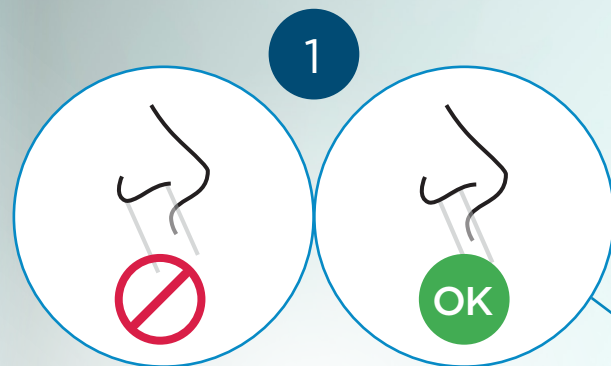


Select appropriate size. Prongs must not create a seal in the nares. A clear gap must be visible around each prong.

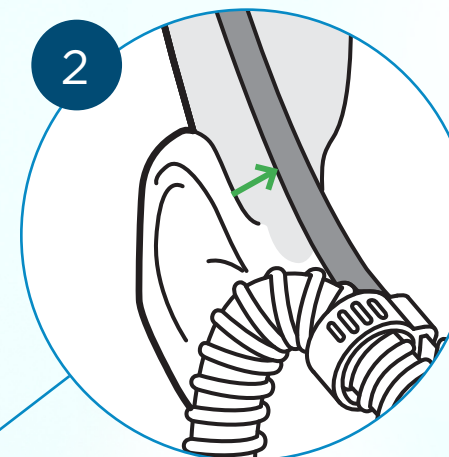


For patient comfort keep the cannula headstrap away from the ears

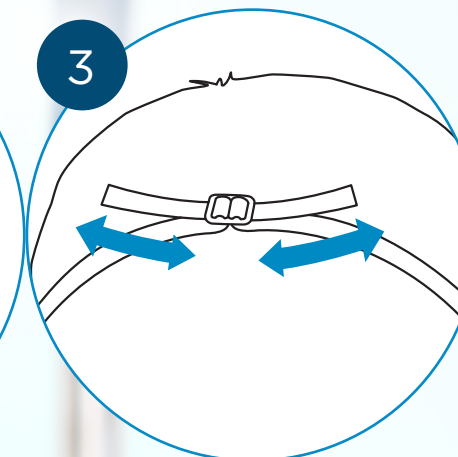


Adjust headstrap to fit. Do not over-tighten.

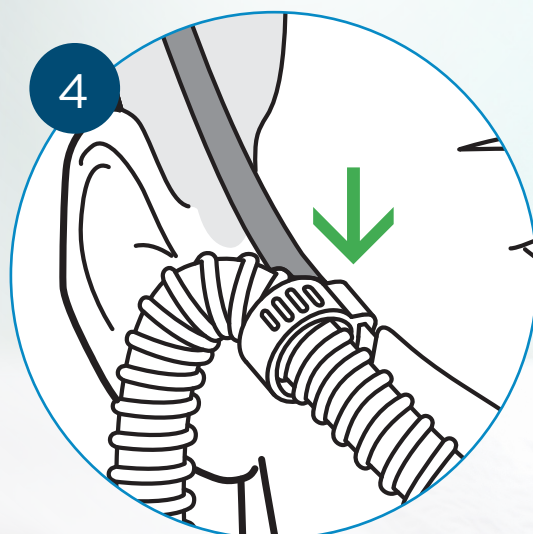


Ensure headstrap clip is attached, to prevent cannula from being pulled out of the nares.

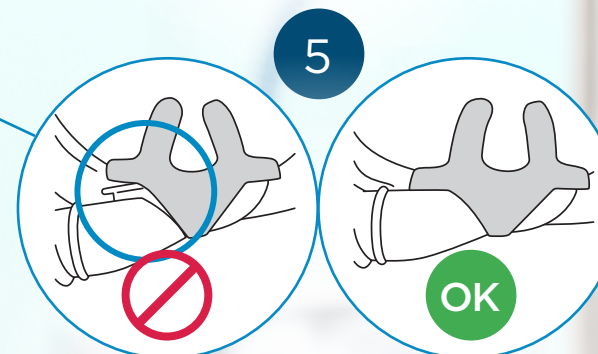

Cannula can become unattached if not used with the headstrap clip.

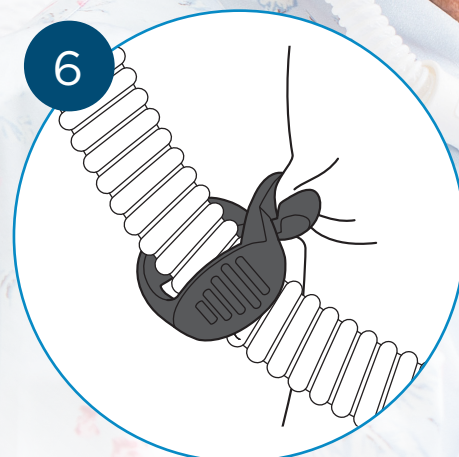

Attach tubing clip to clothing/bedding to prevent cannula from pulling off face.

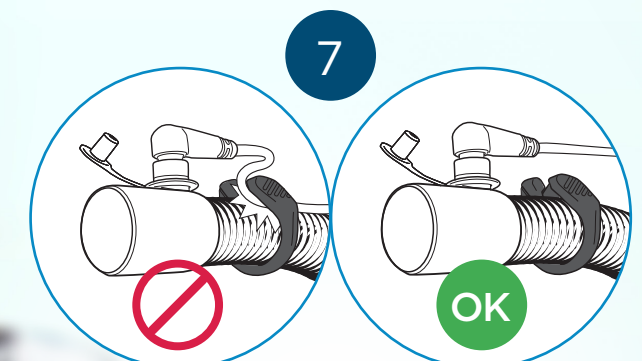

If using an MR850 Heated Humidifier, attach the tubing clip to the breathing circuit but ensure probe cable is not crushed by tubing clip.