
Nasal-Aire® II - Instruction For Use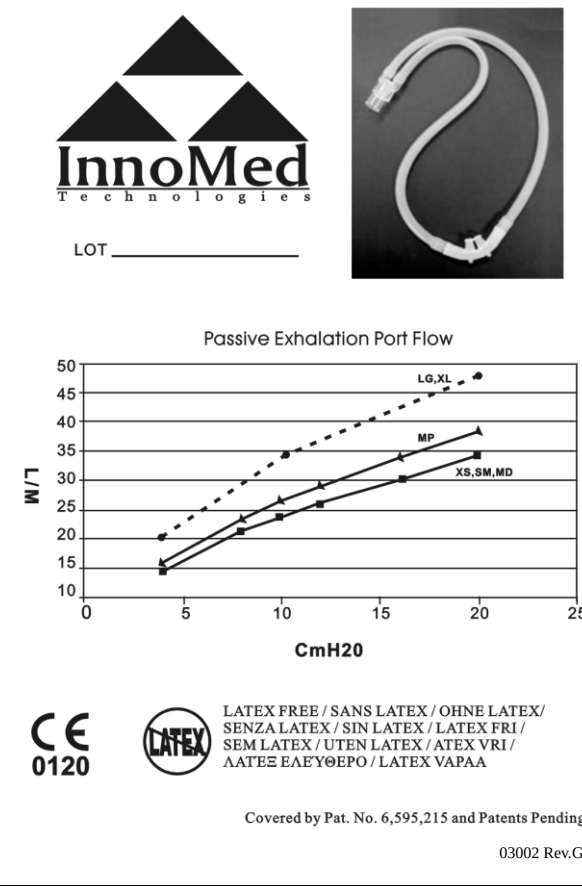

ENGLISH

Homecare Applications

Nasal-Aire II Reusable CPAP Interface is an accessory to positive pressure ventilation devices for patients with Obstructive Sleep Apnea requiring positive pressure breathing therapy (i.e. CPAP, BiLevel). Nasal-Aire II is designed for patient comfort and to facilitate patient compliance, so that the maximum benefit of the prescribed therapy may be derived.

System Components

Nasal-Aire II consists of a soft nosepiece with exhaust ports and two soft silicone nasal cannulas for fitting into the nasal openings. Connected on each side of the nosepiece is a length of tubing. The two lengths of tubing are joined at the opposite ends by a “Y”-shaped coupling, forming a loop with the nosepiece at one end and the “Y” coupling at the other. At the base of the “Y” coupling is a rigid port for connection of the Nasal-Aire II to a positive pressure ventilation device.

Preparation for Use

1. Remove Nasal-Aire II from package.
2. Wash and rinse assembly according to the Cleaning Instructions before use.
3. Complete circuit assembly before use.

Directions for Use

1. With cannula facing nose, place tubing over and around ear with “Y” connector resting on chest.
2. Grasp nosepiece with thumbs and fingers under each nasal insert and position near nostrils.
3. Using thumbs, press inserts into nostrils until base of Nasal-Aire II nosepiece is Flush against the nose.
4. Turn ventilation device on.
5. Attach “Y”-adapter to CPAP tubing.
6. Once airflow starts, adjust Nasal- Aire II to fit for comfort and seal. Periodic adjustments may be necessary.

Neckstrap Attachment

A Neckstrap attachment is standard with all Nasal-Aire II assemblies.

To attach Neckstrap and optional Headgear:

1. Loop one end of the hook and loop fastener tab around Nasal-Aire II tubing on one side.
2. Wrap strap around back of head.
3. Loop other end of hook and loop fastener tab around tubing on opposite side and adjust for comfortable fit.
4. The Nasal-Aire II comes with an optional headgear. To use the optional headgear, slide the Figure 8 Loop onto one end of the headgear. Place headgear on head, secure all the hook and loop fasteners, and position for comfort. Slide the Figure 8 Loop on the headgear towards the back of the head.
5. Disconnect the nasal section from the Nasal-Aire II. With the “Y” swivel coupling on or near chest, place one tube around and behind the ear, slide it through the tubing holder ring that is on the headgear, and pull tubing toward the nose. Repeat for other

- tubing. Reattach nasal section to tubing.
6. Disconnect neckstrap from tubing. Insert neckstrap through the bottom loop section of the Figure 8 Loop. Reattach neckstrap as instructed in steps 1 through 3.
 7. Adjust location of tubing holder rings on headgear as needed for comfort.
 8. The Nasal-Aire II may now be taken off and put back on the head repeatedly without removing the nasal section, headgear, or neckstrap.

Bolo Adjustment

1. Remove hook and loop fastener bolo component from package.
2. Secure bolo around tubing on chest.
3. Adjust bolo position until comfortable.

Cleaning Instructions

1. Disconnect nasal cannula from tubing.
2. Clean all components with mild soap and warm water. Soaking for extended periods of time is not recommended.
3. Shake excess water from interior surfaces of components.
4. A cotton swab may be used to clean openings of tubing, nosepiece and nasal inserts.
5. Set aside to air dry.
6. Reassemble before use.

Testing has shown that the Nasal-Aire II may be cleaned at least ninety (90) times. Nasal-Aire II should be cleaned after each use. Discard when signs of wear are evident. Signs of wear include rips, tears, holes to the cannula or tubing, or visible cracking of any component. Discoloration is normal after several washings. And should not be considered a sign of wear. See your homecare provider for replacement.

Warnings and Precautions

- Federal (USA) law restricts this device to use by or on order of a physician. Use only as prescribed by a qualified physician.
- Do not use this device without instruction from your doctor, home health care professional, or your respiratory therapist.
- Nasal-Aire II must be used only with a positive pressure ventilation device.
- Nasal-Aire II is intended for single-patient use only. Use by multiple patients is to be avoided due to the risk of transmitting infection.
- Do not attempt to sterilize or use disinfectants on this device or any of its components.
- The Nasal-Aire II should not be used unless the ventilation system is turned on and operating. The exhalation ports should never be blocked.

Explanation of warning:

CPAP systems are intended to be used with special nasal cannulas with intentional ports to allow a continuous flow of air out of the nasal cannula. When the CPAP machine is turned on and functioning properly- new air from the CPAP machine flushes the exhaled air through the intentional leak port. However, when the CPAP is not operating, enough fresh air will not be provided through the nasal cannula, and exhaled air may be rebreathed. Rebreathing of exhaled air for longer than several minutes can in some cases lead to suffocation. This warning applies to most models of CPAP machines.

- Nasal-Aire II is not intended for use on ventilation systems with pressure alarms. The device can deactivate the alarm.
- Some users may experience skin redness. If this occurs, consult your doctor.
- Do not expose this device to extremes of temperature. Store the Nasal-Aire II at temperatures between 0°C (32°F) and 54°C (130°F).

Please direct any questions concerning your Nasal-Aire II accessory to your healthcare professional OR Innomed Technologies.

Reorder information

Extra Large Nasal-Aire II System	XL306
Large Nasal-Aire II System	LG305
Medium Plus Nasal-Aire II System	MP304
Medium Nasal-Aire II System	MD303
Small Nasal-Aire II System	SM302
Extra Small Nasal-Aire II System	XS301

Nasal-Aire II is brought to you by

Manufactured for: InnoMed Technologies
Corporate: 6601 Lyons Rd., Suite B 1-4, Coconut Creek, FL 33073, USA
Tel: (800)-200-9842

Manufactured by: MODERN MEDICAL EQUIPMENT MFG. LTD.
5fl, 35 Tai Lin Pai Rd., Kwai Chung, Hong Kong.
Tel: (852) 2420 9068 Fax: (852) 2481 1234
European Representative: Symbol Medical GmbH
Schwarzschildstraße 6, D-12489 Berlin, Germany.
Email: mmeq_eu@modernmedical.com.hk

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Nasal-Aire II besteht aus einem weichen Nasenteil mit Atemdurchlässen und zwei weichen Silikon-Nasenkanülen zum Einführen in die Nasenöffnungen. Angeschlossen an jede Seite des Nasenteils ist ein längerer Schlauch. Die zwei Schläuche werden an den egenüberliegenden Enden mit einem “Y”-förmigen Verbindungsstück verbunden und formen so eine Schleife mit dem Nasenteil auf der einen Seite und dem “Y”-förmigen Verbindungsstück auf der anderen Seite. An der Unterseite des “Y”-förmigen Verbindungsstücks befindet sich ein steifer Durchlass für die Verbindung des Nasal-Aire © II mit einem positiven Druckbeatmungsgerät.
Valmistaja: MODERN MEDICAL EQUIPMENT MFG. LTD.
Edustaja Euroopassa: Symbol Medical GmbH