

Step 1 – Cleanse the skin



[CLICK TO LEARN MORE](#)




Remove excess makeup, skin moisturizer, or cleansers. Allow the skin to dry.

Step 2 – Open and remove



Slowly peel back the upper layer of protective packaging and remove the NozeSeal adhesive strip starting from the center.

Step 3 – Peel and fold

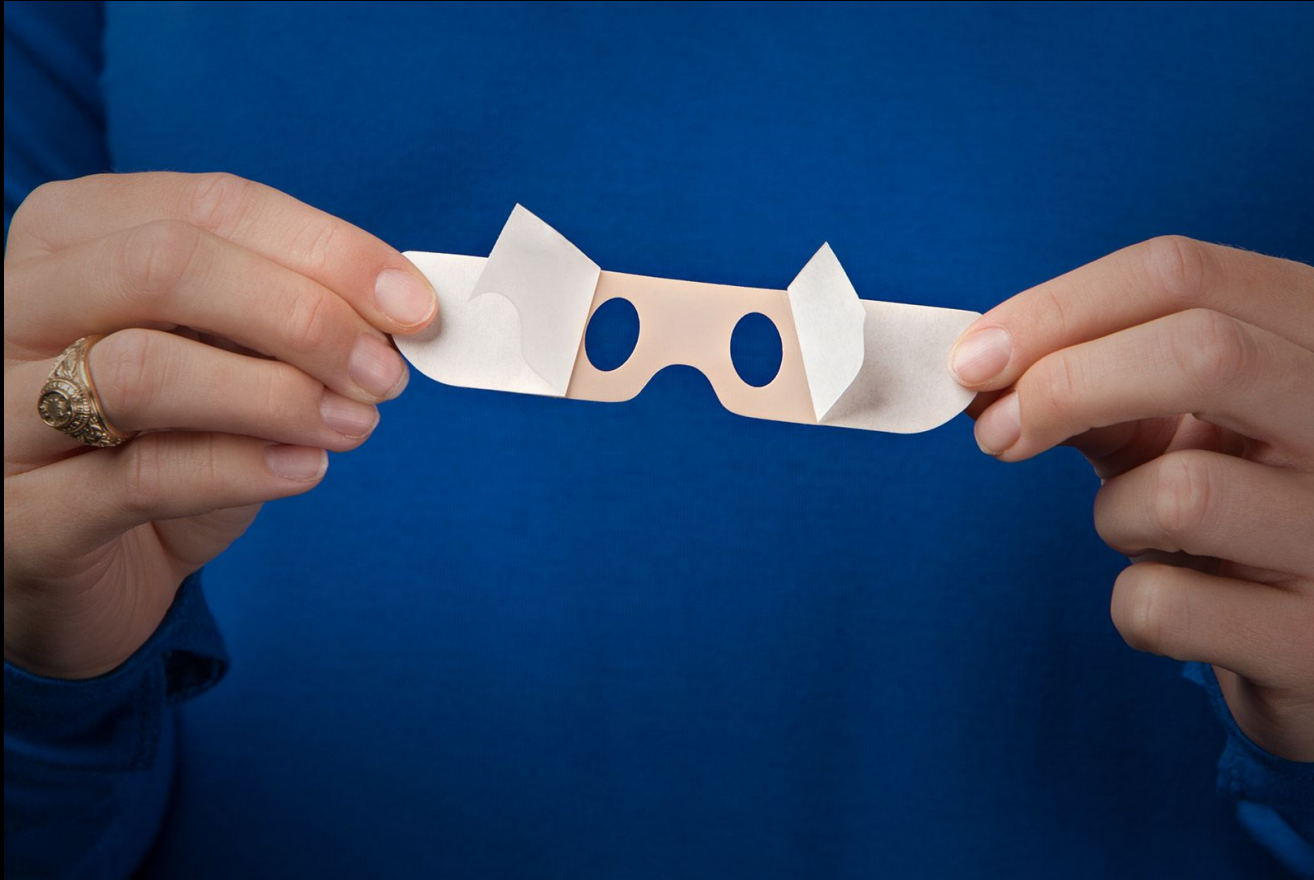

Peel back the protective layer and fold to reveal the openings.

Step 4 – Align




Align the hole pattern of the NozeSeal adhesive strip with the hole pattern of the nasal pillow.

Step 5 – Squeeze, insert, and pull



Squeeze the nasal pillow, insert the outer edge, and pull through the hole.

Step 6 – Hold, peel, and adhere

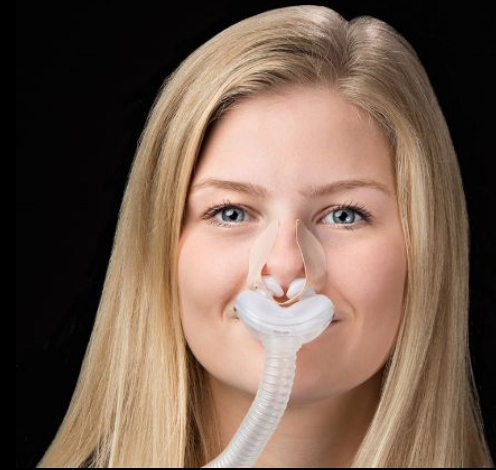

Hold the nasal pillow to your nostril, peel the protective layer, and adhere to your nose.
(If you also are using optional headgear, please refer to your manufacture's instructions.)

Step 7 – Connect



Connect the mask interface tubing to the long tubing coupled to your machine.

Step 8 – Sleep



Get a great night's sleep!

Step 9 – Disconnect



Disconnect the mask interface tubing from the long tubing coupled to your machine.

Step 10 – Remove



From the outermost tips, slowly peel away the NozeSeal adhesive strip.
For best adhesion and hygiene, discard after a single use.