

# ACCESSORY KIT

(FOR USE WITH P2, P2-E, P2-E6, AND P2-E7)



## OPERATING INSTRUCTIONS

### Please read and understand instructions before using Accessory Kit.

The Accessory Kit is designed specifically for use with the P2, P2-E, P2-E6, and P2-E7. Always follow the instructions included with the Accessory Kit and Portable Oxygen Concentrator.

Batteries can be charged using one of the following options:

- The External Battery Charger
- The AC Adapter
- The DC Adapter



### 2.5 Hours to Fully Charge

1. Plug one end of the AC power cord into a wall outlet and the other end into the charger.



2. Slide the battery into the charger until you hear it click into place.



3. Flashing green light (at the top of the charger) indicates that the battery is charging.

4. When all five green lights are solid, the battery is fully charged.

5. Disconnect from power. Press down on the latch on the bottom of the charger and slide the battery out.



### IMPORTANT

Use Power port #1 for AC Adapter (P2ACA-2)  
Use Power port # 2 for AC Adapter (P2ACA-1)



Rhythm Healthcare, LLC  
Clearwater, FL 33762

Email: [contactus@rhythmhc.com](mailto:contactus@rhythmhc.com)  
Website: [www.rhythmhc.com](http://www.rhythmhc.com)

V3: 4/29/25