

PAPTIZER 360 CPAP MICROBIAL REDUCTION DEVICE

FDA 510(k) CLEARED

SKU: T5L360



The Paptizer 360 is intended to be used as an adjunct to reduce bacterial populations on compatible home-use CPAP mask cushions after cleaning. Utilizing a 40-light UVC LED array, it offers a simple, targeted 30-minute bacterial reduction cycle.

HOW IT WORKS

The Paptizer 360 utilizes UVC LED light technology to safely and effectively reduce microorganisms. By pressing the START button, users initiate a precise 30-minute bacterial reduction cycle. An in vitro 2-log (99%) bacterial reduction has been demonstrated after a complete processing cycle for the following most commonly found CPAP pathogens: *Staphylococcus epidermidis*, *Staphylococcus aureus*, *Klebsiella pneumoniae*, *Pseudomonas aeruginosa*, *Escherichia coli*, and *Streptococcus pyogenes*.

- **Microbial Reduction:** UVC process for microbial reduction, not ozone.
- **Simple Operation:** Features an easy-to-read LCD display that counts down the 30-minute cycle.
- **Focused Processing:** Designed to process one item at a time. The mask cushion must be placed facing upward on the tray to ensure optimal microbial reduction.
- **Safety Sensors:** If the lid is opened during operation, the UVC LED lights automatically turn off to prevent exposure to eyes or skin.



Note: This device must not be used to replace the cleaning procedures as recommended by the CPAP mask manufacturers. Pair with CPAP Soap for the ultimate cleaning system.

INCLUDED PARTS

- Paptizer 360 Unit
- Power Supply
- Quartz Glass Tray
- Suction for Tray
- Instructions for Use

SPECIFICATIONS

Input: 18V / 1.5A

Power Rating: 27W

Input Interface: Type C USB

UVC LED Quantity: 40

UVC Wavelength: 260–280 nm

LED Lifespan: Up to 10,000 hours

Dimensions: 11.89" x 10.59" x 6.06" / 302 x 269 x 154 mm

Weight: 4.975 lbs. with quartz glass plate

Unit Weight: 4.069 lbs.

Quartz Glass Plate: 0.906 lbs.

Operating Temp: 41 to 104°F (+5 to +40°C)

Storage Temp: -13 to 158°F (-25 to +70°C)

Humidity Limits: 15% – 90%

Emissions/Immunity Tested to Group 1, Class B

2 year warranty



[CLICK TO LEARN MORE](#)

