

PAPTIZER 360 QUICK START GUIDE

1

CLEAN IT FIRST



[CLICK TO LEARN MORE](#)



Remove the headgear, the frame, and the tubing. Wash the cushion per the mask manufacturer's instructions and let it dry.

ONE CUSHION, FACING UP

2



Place a single cushion on the quartz glass tray with the opening facing upward, so the LEDs reach the surfaces that touch skin.

3

START THE CYCLE



Close the lid, press ON, then press START. The LCD counts down thirty minutes.

LEAVE IT ALONE

4

When the cycle ends, the unit shuts off. There is nothing to rinse, nothing to wipe, and nothing to reorder. Your mask is ready to use immediately. No product downtime needed.



**SCAN OR VISIT
WWW.LIVILITI.COM
TO REGISTER THE
WARRANTY:**



info@liviliti.com