

PARI lends a hand with
the **one-handed**
operation aid



The patented one-handed operation aid helps parents and caregivers deliver treatments to babies using just one hand, so you can hold your child safely during inhalation.

Scan the QR code below for a
short video on how to use the
PARI VORTEX® and accessories



PARI VORTEX® for patients of all ages



**VORTEX with
Pediatric Mask, Sm**
0 - 1.5 years



**VORTEX with
Pediatric Mask, Med**
1.5 - 4 years



**VORTEX with
mouthpiece**
4 years and up

VORTEX® baby and child masks— comfortable and efficient

- **A comfortable fit** – The soft mask fits seamlessly to the contours of the face
- **Efficient inhalation** – Robust seal even when applied gently to the face
- **Super easy hygiene control** – Easy to clean, dishwasher safe, and boilable



PARI Respiratory Equipment, Inc.
2412 PARI Way, Midlothian, VA 23112
Tel: 1-800-FAST-NEB (327-8632)
product.info@pari.com | www.pari.com

© 2024 PARI Respiratory Equipment, Inc. 051D7005 Rev E US-VOR-1800006 v4 09-24



PARI VORTEX® Antistatic Holding Chamber


QUICK GUIDE

Aluminum Chamber.
Less Plastic.
Better Delivery.



www.pari.com/vortex

As easy as 1, 2, 3: Assembly & inhalation with... mouthpiece

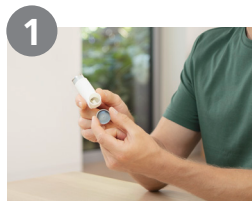

1 Remove the inhaler cap, shake vigorously



2 Insert the inhaler into the receptacle on the back of the holding chamber



3 Remove the protective cap on the mouthpiece



4 Exhale, then place the mouthpiece between teeth and close lips around it



5 Press down to actuate the inhaler



6 Breathe in and out slowly and deeply

Did your physician prescribe several sprays per use? Repeat steps 5 and 6.

0 - 18 months: 4 - 6 breaths, **>18 months:** 2 - 4 breaths

mask and one-handed operation aid



1 Remove the inhaler cap, shake vigorously



2 Push the one-handed operation aid onto the holding chamber



3 Fit the inhaler into the connecting ring and click the one-handed operation aid into place



4 Remove the protective cap and attach the mask



5 Apply to face over nose and mouth creating a good seal



6 Actuate the inhaler, breathe in and out slowly and deeply

Did your physician prescribe several sprays per use? Repeat step 6.

0 - 18 months: 4 - 6 breaths, **>18 months:** 2 - 4 breaths

Cleaning and disinfection: Step-by-step in the video

Thanks to the durable aluminum chamber and removable mouthpiece, the VORTEX® and masks can be boiled, disinfected, and washed in the dishwasher.

