



Contact Transcend Inc.  
Corporate headquarters  
Transcend Inc.  
Email  
customersupport@mytranscend.com  
Web  
http://www.MyTranscend.com  
Telephone  
651.621.1800  
Toll-free telephone  
877.621.9626  
Fax  
651.204.0064  
Transcend Inc.  
103 Osborne Road NE Fridley,  
Minnesota 55432 USA

Notices  
Notice — the information contained in  
this document is subject to change  
without notice.  
Trademark — The moon logo, Transcend Micro,  
PowerAway, and Transcend are trademarks  
and/or registered trademarks of Transcend Inc.  
Copyright — © Copyright 2022 Transcend Inc.;  
all rights reserved.

LED Indicators

- The four LED lights on the battery indicate the power level. Press the fuel button to determine the battery status.



| Power Connection | Power LED | State of Charge | LED 1    | LED 2    | LED 3    | LED4     |
|------------------|-----------|-----------------|----------|----------|----------|----------|
| Disconnected     | Off       | Any             | Off      | Off      | Off      | Off      |
|                  | On        | < 25%           | Blinking | Off      | Off      | Off      |
|                  | On        | 26%-50%         | On       | Blinking | Off      | Off      |
| Connected        | On        | 51%-75%         | On       | On       | Blinking | Off      |
|                  | On        | 75%-99%         | On       | On       | On       | Blinking |
|                  | On        | 100%            | Off      | Off      | Off      | Off      |

Caring for the Battery

- DO NOT immerse battery in water or cleaning solution.
- Wipe the surface with a dry, lint-free cotton cloth.
- For long term storage, which is any duration of time greater than three months, the battery should be:
  - Stored at a temperature above -20C/-4F but below 20C/68F.
  - Stored in a dry location with a relative humidity below 70%.
  - Stored with a 30-40% charge

Please refer to the Transcend Battery User Guide for more information.  
[www.mytranscend.com/user-guides](http://www.mytranscend.com/user-guides)



CLICK TO LEARN MORE

## Transcend Micro PowerAway® Battery Quick Guide



The Transcend PowerAway battery is an optional power source that can be used as a primary power supply, or in conjunction with AC power supply as backup power for uninterrupted therapy in the case of a power outage.

The Transcend PowerAway battery is approved for use with the Transcend Micro device.

## Instructions


### Preparing Battery for First Use

Before first use, the Transcend Micro PowerAway Battery needs to be fully charged.

1. Remove the battery from its packaging.
2. Plug the end of the AC Power Supply cord into the PowerAway battery.
3. Plug the AC Power Supply into an AC outlet.


The battery will begin to charge automatically. It may take up to six (6) hours to charge the battery.

### Connecting with the Laptop style Power Supply

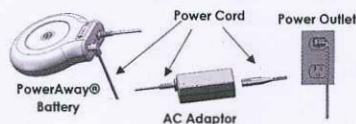

OR

### Connecting with the Wall Mount style Power Supply

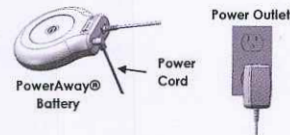

Once the battery is fully charged, it is ready for use with your Transcend CPAP device. A fully charged battery is indicated when the LED lights on the battery turn green after pressing the fuel button.

### **NOTE:**

Do not connect the battery to your Transcend CPAP device during the initial charge. The CPAP draws power from the battery while connected. If the battery is not used in line with the AC power supply, unplug the battery from the CPAP when not in use.

### Using Battery

To use the Transcend PowerAway battery, connect the battery outlet cable to your compatible Transcend CPAP device by inserting the barrel connector from the battery into the power jack on the Transcend CPAP device.

