

Air Filter

There are 2 kinds of air filters, the black Reusable Air Filter and the optional white Disposable Ultra-Fine Filter.

Reusable Air Filter (Black)

- Do **NOT** run the device without the Reusable Air Filter in place.
- Check the Reusable Air Filter and replace it at least every 6 months, or if there are any holes or blockages due to dirt or dust.

Optional Disposable Ultra-Fine Filter (White)

- Check the Disposable Ultra-Fine Filter and replace it at least every 2 weeks, or if there are any holes or blockages due to dirt or dust.
- The Disposable Ultra-Fine Filter is NOT washable or reusable.
- The Disposable Ultra-Fine Filter must be used together with the Reusable Air Filter. Do NOT use it alone. When installing, place the Reusable Air Filter into the Filter Cap first, and then place the Disposable Ultra-Fine Filter on top.



Cleaning and Replacing Guidelines

Regularly cleaning and replacing the accessories is essential for hygiene and ensuring that your therapy remains as clean, comfortable, and effective as possible.

Accessory Component	Cleaning Schedule	Cleaning Method	Replacing Schedule
Nasal / Pillows Cushion <sup>1</sup>	Daily	A. Wash in a solution of warm water and mild dishwashing liquid or soapy water  B. Rinse thoroughly with clean water  C. Air dry away from direct sunlight	Twice per month
Full Face Cushion <sup>1</sup>	Daily		Every month
Mask Frame <sup>1</sup>	Daily		Every 3 months
Mask Headgear <sup>1</sup>	Weekly		Every 6 months
Air Tubing	Weekly		Every 3 months
Water Chamber of Humidifier	Weekly		Every 6 months
Reusable Air Filter	Every 2 weeks	A. Wash with running tap water  B. Air dry away from direct sunlight	Every 6 months
Optional Disposable Ultra-Fine Filter	NOT washable or reusable		Every 2 weeks


<sup>1</sup>The guidelines provided are for React Health masks. When using a mask from another manufacturer, consult the guidelines provided with the mask.

G3 X APAP/BPAP Quick Operation Guide






- Home button for returning to the previous menu or main interface.
- Start/Stop button for starting or stopping the therapy.
- Knob for navigating and changing options by pressing and turning.
- Air outlet for connecting the breathing tube/heated breathing tube.
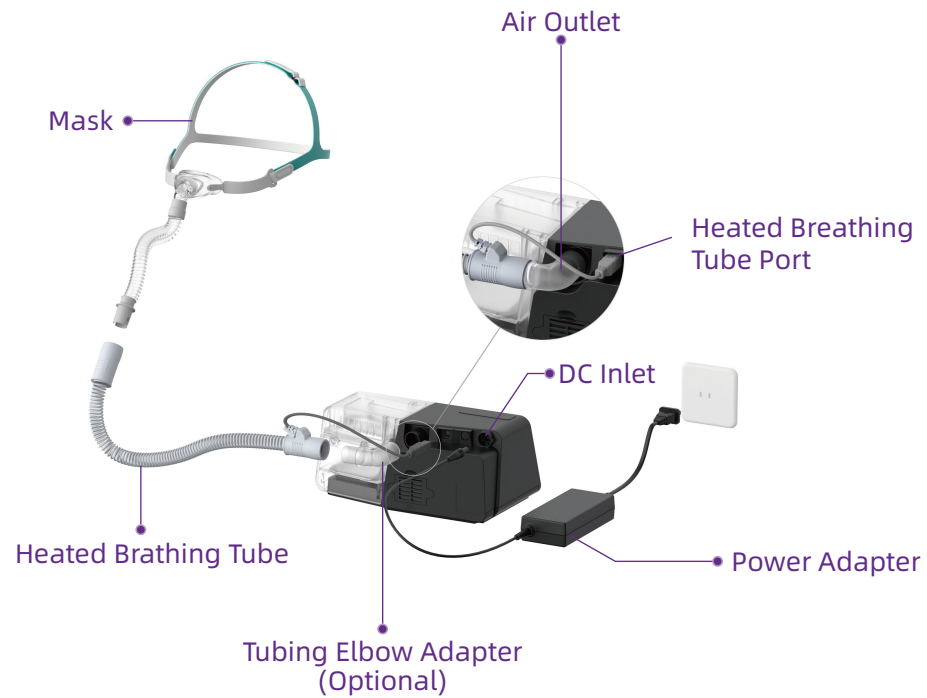
## Setup



## General Guidelines

### Device



- To start the device, click the  Start / Stop Button.
- To stop the device, click the  Start / Stop Button for APAP devices, or press and hold the  Start / Stop Button for 2 seconds for BPAP devices.
- Keep the air inlet far away from fire, open flames (such as candles) and aromatic products (such as incense).

### Humidifier

- Press down on the top of the water chamber in the location shown below to remove or install it.

To add water:

- First, remove the water chamber from the device.
- Do **NOT** fill the water chamber while it is still attached to the device. Getting water into the interior of the device can cause equipment damage.

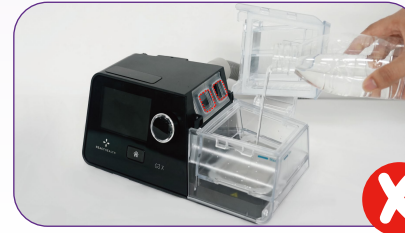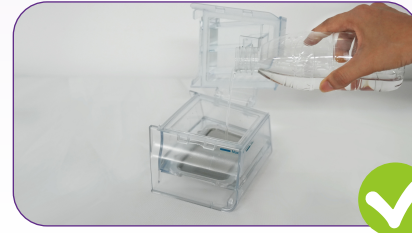

- **Empty** the water chamber before moving or transporting the PAP device to prevent getting water into the interior of the device.
- If there is condensation in the air tubing (and/or mask):

- Drain the condensation from the mask end. Avoid raising the tubing to drain condensation back in the device, as this may cause damage to the device.



- Change the water every day. Distilled water is recommended, or filtered or cooled boiled water. Do not use mineral water.

### Breathing Tube or Heated Breathing Tube

- When connecting or disconnecting the tubing from the device's air outlet:

- Pull the connector at the end of the tubing.
- Do **NOT** pull on the tubing itself.



- For heated breathing tube, avoid squeezing the cable part of the heated breathing tube.

