

How to fit the AirTouch F30i Comfort



With one hand holding the bottom of the cushion and the other on the frame, position the mask in front of your face.



Place the cushion under your nose and ensure it sits against your face. With the Resmed logo on the headgear facing up, pull the headgear and frame over your head.



Bring the lower headgear bands under your ears and attach the magnetic clips\* to the frame connectors.




Undo the fastening tabs on the upper headgear bands and pull evenly.



Repeat step four with the lower headgear bands.



Connect the air tubing from the device to the mask elbow.

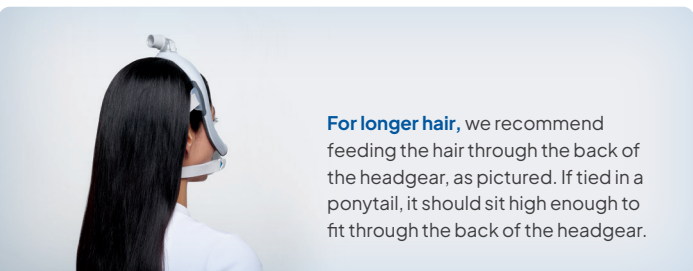


For longer hair, we recommend feeding the hair through the back of the headgear, as pictured. If tied in a ponytail, it should sit high enough to fit through the back of the headgear.

Crafted for softness and comfort to elevate the sleep experience

The AirTouch F30i Comfort is designed to offer patients soft, breathable comfort from the first fit to the last moment before bed. Its adaptive seal is designed to stay secure as patients move, while the top-of-head tube connection gives patients the freedom to sleep in their preferred position. The quiet venting design supports a more restful environment for patients and their bed partners, and the mask is designed to be easy to wear so that it can easily become a natural part of a bedtime routine — helping patients feel at ease and confident staying on therapy.



Scan the QR code or contact your Resmed sales representative to get going.

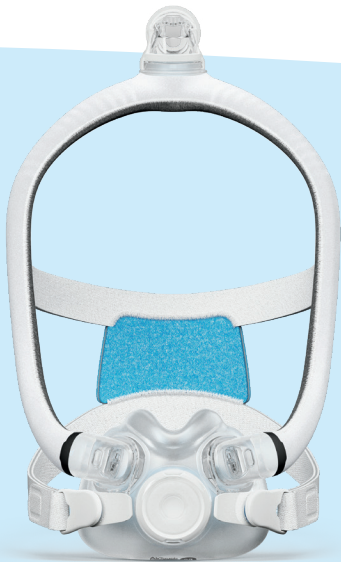

\* The mask contains magnets that may interfere with certain implants or medical devices. Please refer to the User Guide for complete labeling information, including magnet contraindications and warnings.

Patients are encouraged to speak with their treating clinicians for any clinical symptoms or medical conditions.

<sup>1</sup> Resmed internal testing for the sound power of AirTouch F30i mask with QuietAir vent, ID: A5637990

<sup>2</sup> Resmed internal testing for the moisture-wicking properties of fabric, ID: A5498268.

Distributed by Resmed Corp., 9001 Spectrum Center Blvd., San Diego, CA 92123 USA. AdaptiSeal, AirFit, AirTouch, ComfiSoft, and QuietAir are trademarks of the Resmed family of companies. For patent and other intellectual property information, see Resmed.com/ip. © 2025 Resmed. 10114460/1 2025-10

Softness at every touch. Confidence in every fit.

The Resmed AirTouch™ F30i Comfort full face mask





# Designed for all-around comfort

The AirTouch F30i Comfort is designed to surround patients in softness with a fully fabric-wrapped frame and a ComfiSoft™ cushion that fuses breathable fabric with flexible silicone. Together, they create a mask that feels gentle all around, while AdaptiSeal™ technology adapts to facial contours for a secure fit.



**Fabric-wrapped flexible frame**  
Available in three sizes, the frame adjusts easily to different facial profiles and head shapes for an accurate fit the first time, saving time in setup.


**Magnetic clips\***  
Snap on and off for easy fit, adjustment and removal.

**Flexible and UltraCompact full face cushion**  
The flexibility and under-the-nose design reduce facial coverage, creating a lighter feel on the face. Available in three sizes and interchangeable with the AirFit™ F30i and AirFit X30i.

**A Resmed-first innovation**  
The combination of our innovative ComfiSoft™ silicone-coated fabric cushion with AdaptiSeal™ technology that contours to the face without any rigid plastic delivers maximized comfort and secure seal performance.



**Top-of-the-head tube connection and quick-release elbow**  
Allows patients to sleep in the position they want via 360-degree elbow rotation, while also enabling them to easily connect and disconnect the device tubing.

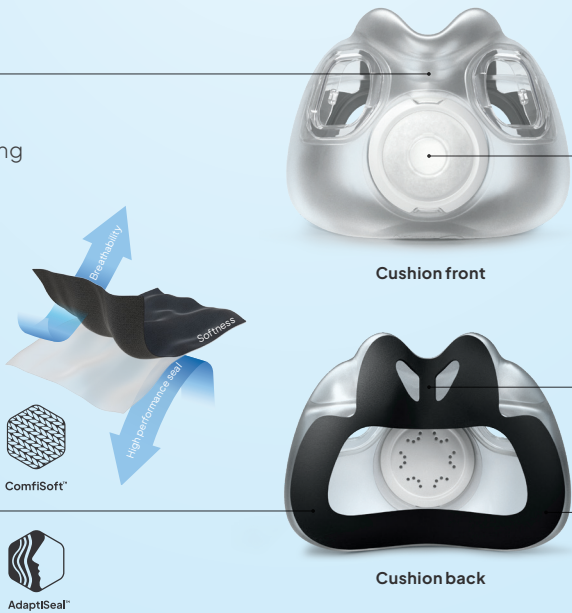

**QuietAir™ venting**  
Measured 20 decibels, quiet and diffused venting minimizes disturbance to both patients and bed partners.<sup>1</sup>

**Nasal septum bridge**  
Designed to hold the shape of the cushion to create a comfortable and reliable seal.

**Soft, breathable and moisture-wicking**  
Soft fabric helps to keep the nose and mouth area cooler and less sweaty for ultimate skin comfort.<sup>2</sup>



**Complete mask system**  
62442  
62443  
62444  
62445  
62446

**UltraCompact full face cushion**  
62489  
62490  
62491

**QuietAir vent**

**Fabric-wrapped frame with elbow**

**F30i/X30i headgear**  
63372

**Magnetic clips\***  
63475

## AirTouch™ F30i Comfort

Complete mask system (includes mask frame with elbow, full face cushion and headgear)	
Product name	Product code
AirTouch F30i Comfort mask system - M/SML	62442
AirTouch F30i Comfort mask system - SW/STD	62443
AirTouch F30i Comfort mask system - M/STD	62444
AirTouch F30i Comfort mask system - L/STD	62445
AirTouch F30i Comfort mask system - M/LGE	62446

Headgear (includes magnetic clips*)	
Product name	Product code
F30i/X30i headgear	63372

UltraCompact full face cushion with QuietAir vent (ComfiSoft™ cushion with AdaptiSeal™ technology)	
Product name	Product code
AirTouch F30i cushion - SW	62489
AirTouch F30i cushion - M	62490
AirTouch F30i cushion - L	62491

**Cushion Sizes**  
S=Small | SW=Small-wide | M=Medium | L=Large

**Frame Sizes**  
SML=Small | STD=Standard | LGE=Large

Live life in high Res

