


iQ[®]2 Nasal Mask

Quick Start Guide

[CLICK TO LEARN MORE](#)

Before putting on your mask, adjust the back of the headgear according to your head size. Start at medium to large, and work your way left or right.

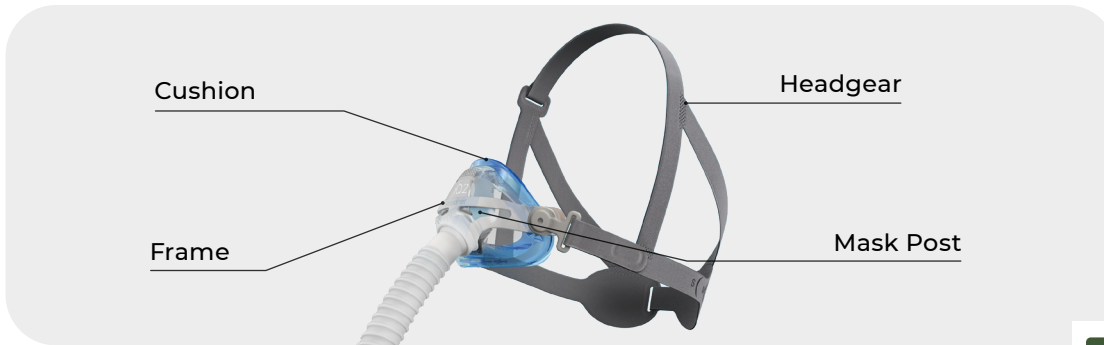

Detach one easy-release magnetic clip from the mask, slip the mask over your head, then reattach the magnetic clip.



Adjust the top strap to raise or lower the mask to a comfortable position. Make sure your ear is inside the triangle of the headgear.



Adjust the left and right straps to ensure the mask sits comfortably between the bridge of the nose and the upper lip.